

Bladder



Refer to user manual before use.

SEE FOR YOURSELF

QUICK START GUIDE

Getting Started

1



On/Off button

Press for 1 second to turn on or off



Green The device is on



Flashing Power saving mode



Amber Low battery

2



'Start'

Begin scanning

'Start + Patient'


Enter patient details before scanning

'Review'

Review previous scans

Scanning

3

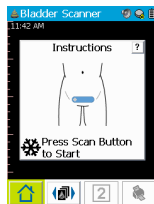

Positioning

The probe position is displayed in **blue**.

Help Button

Using the help button  will give more scanning instructions.

4



Use of Probe

Apply gel. Place the probe 1cm above the pubic bone with silver orientation dots toward the patient's right hand side.

Angle 10 degrees toward the patient's feet.
Scan from the bottom to the top of the bladder.

Scanning

5




Record the Scan

To begin scanning, press the scan button  on either the probe or display.

Once you have obtained the largest volume, press the scan button  to stop scanning. Images and measurements are automatically saved.

6

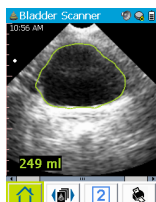

Reading the Scan

The bladder will be outlined as you scan.

The volume measurement is automatically updated as you scan and is located in the bottom left corner of the display.

This measurement may be repeated as needed by pressing the scan button again.

Save & Review

7



Save Scan

Select the button  to save, end and close the scan.



Second Scan

Select the  to perform a second scan in the vertical orientation for improved accuracy. Image two is a combined volume of both images.

8



Review

Select the  to review previous images of the current patient.

To view previous patient examinations from the home screen, select the 'Review' button.